



ALL MARLUX CURTAINS ARE 100% RECYCLABLE.
 Marlux offers a recycling service* to its customers (subject to conditions); which includes the collection and ethical recycling of used product.
 For more information regarding the Enviro Bag Service please contact Marlux.
 *UK only.




FAST-FIT DISPOSABLE PRIVACY CURTAIN

DID YOU KNOW ?
 MARLUX Disposable Curtains are available for customisable printed messaging.



OR

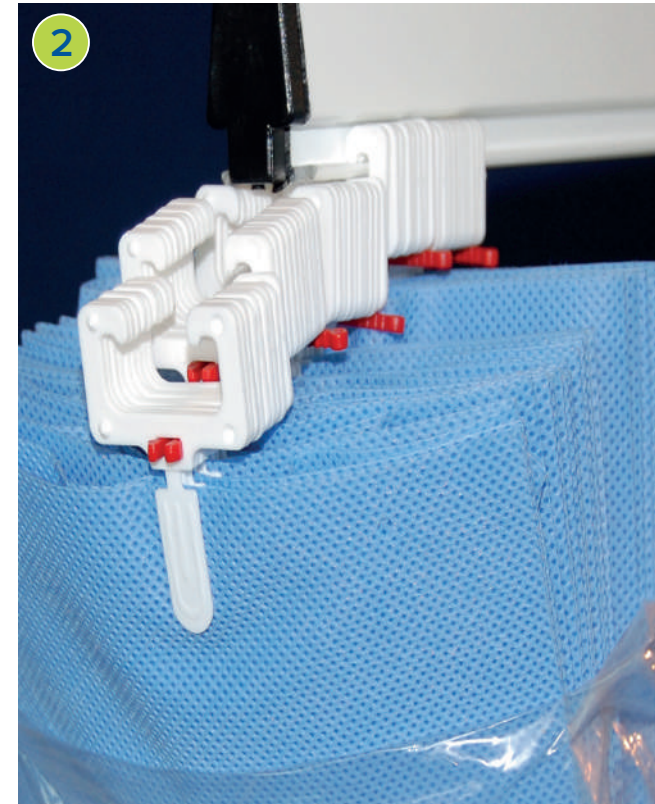

OR



DEPENDING ON TRACK SYSTEM:

- 1A** Push the plunger up. Remove the used curtain and wipe track with a silicone wipe.
- OR**
- 1B** Remove the stainless steel retaining pin. Remove the used curtain and wipe the track with a silicone wipe.

- 2** Open the plastic sleeve at the hook end. Do not fully remove the plastic sleeve at this stage.
 Simply slide the prefitted **FAST-FIT** hooks onto the curtain track.

DEPENDING ON TRACK SYSTEM:

- 3A** Pull plunger down and then break red tab to release curtain gliders.
- OR**
- 3B** Replace stainless steel retaining pin and then remove red tab to release the curtain gliders.

- 4** Remove the plastic sleeve completely. Write the curtain installation date in the area provided.
 To change the curtain repeat from step 1 with a new Marlux curtain.

MARLUX MEDICAL LTD
 Bourton Industrial Park,
 Bourton on the Water, Gloucestershire
 GL54 2HQ, United Kingdom

T: +44 (0) 1451 823 398
E: sales@marluxmedical.com
www.marluxmedical.com

