




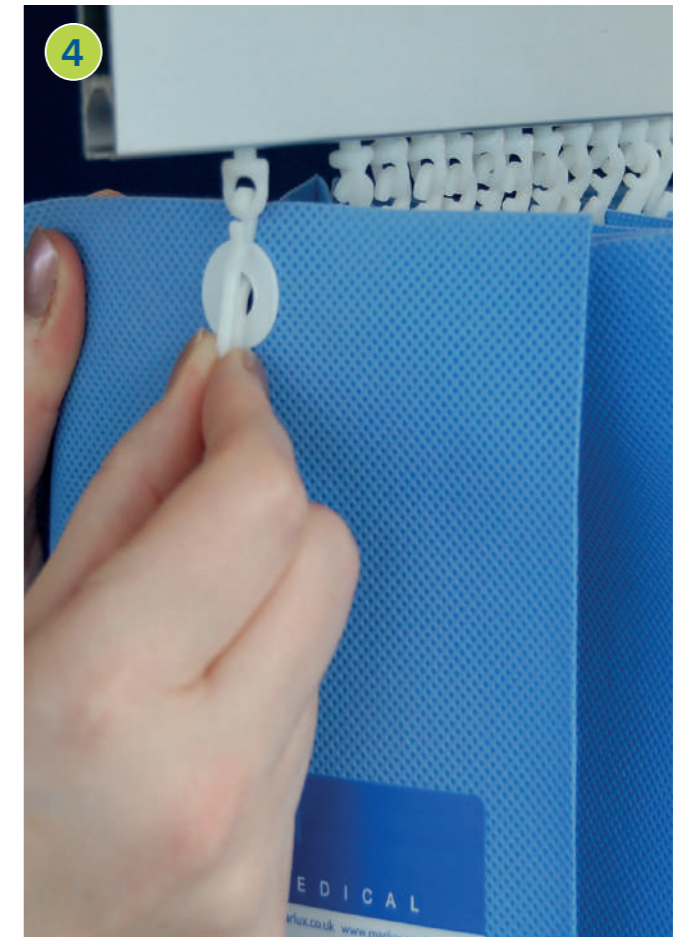
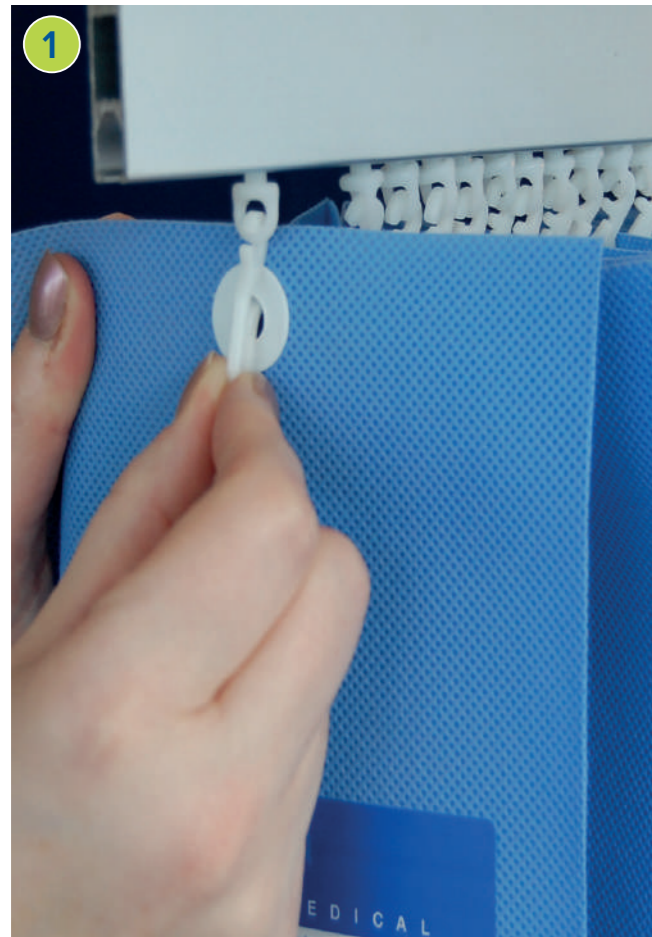
ALL MARLUX CURTAINS ARE 100% RECYCLABLE.
 Marlux offers a recycling service* to its customers (subject to conditions); which includes the collection and ethical recycling of used product.
 For more information regarding the Enviro Bag Service please contact Marlux.
 *UK only.



UNIVERSAL

DISPOSABLE PRIVACY CURTAIN

DID YOU KNOW 
 MARLUX Disposable Curtains are available for customisable printed messaging.



1 Open the plastic sleeve at the eyelet end. Do not fully remove the plastic sleeve at this stage.
 Insert the existing curtain track hook into the curtain eyelet hole.

2 Remove the plastic sleeve completely.

3 Write the curtain installation date in the area provided.

4 To change the curtain simply un-hook the eyelets, then repeat from step 1 with a new Marlux curtain.

**MARLUX™
 FAST-FIT**

Our MARLUX FAST-FIT curtains can be taken down and refitted in 2-3 minutes and will work with most standard tracking. Existing tracks can be used, by simply converting them easily with Adapt-a-Track. Contact us for more information.

MARLUX MEDICAL LTD
 Bourton Industrial Park,
 Bourton on the Water, Gloucestershire
 GL54 2HQ, United Kingdom

T: +44 (0) 1451 823 398
E: sales@marluxmedical.com
www.marluxmedical.com

